

Clinicare

Alcohol Free

NEW Formula
Kills COVID-19

Direction for use

Composition	Benzethonium Chloride 0.40% w/v, Cetrimide 0.45% w/v, Polyaminopropyl biguanide 0.3% w/v
Uses	Clinicare Alcohol Free Hospital Grade Disinfectant solution is a highly effective cleaner and disinfectant. It has been developed for use in hospital operating rooms, isolation areas, neonatal units and other critical care areas where environmental control of cross contamination is important.
Advantages	1 minute kill time BACTERICIDAL, 30 seconds kill SARS-CoV-2(COVID-19) - Alcohol Free - Broad Spectrum, fast acting 2 in 1 disinfectant and decontaminant cleaner - Formulated and tested for use in healthcare and critical environments - Safe for use on all surfaces including plastics, metals, ceramics, leather, vinyl, etc.
Active Against	<i>Pseudomonas aeruginosa</i> , <i>Escherichia coli</i> , <i>Staphylococcus aureus</i> , <i>Enterococcus hirae</i> , <i>Proteus vulgaris</i> , <i>Salmonella enterica</i> , <i>VRE faecium</i> , <i>Klebsiella pneumoniae</i> , <i>Acinetobacter baumannii</i> , SARS-CoV-2 (COVID-19), <i>Candida albicans</i> TESTED UNDER DIRTY CONDITIONS
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.
Precautions	Do not swallow. Avoid contact with skin and eyes. Use undiluted. If visible soiling is evident, surfaces should be pre-cleaned / decontaminated before disinfection.
Clinical Procedure	For Pre-cleaning / Decontamination of Surfaces Apply Clinicare Alcohol Free Hospital Grade Disinfectant directly to the contaminated surface. Physically wipe surface clean using a Clinicare towelette. Discard any towelette used to decontaminate the area. For Disinfecting Surfaces Apply Clinicare Alcohol Free Hospital Grade Disinfectant directly to the pre-cleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain wet for 1 minute under ambient conditions. Allow surface to air dry or, if required, wipe or rinse clean after 1 minute. When preparing surfaces only use Clinicare towelettes or a lint free cloth. Discard any used towelettes after use.



Activity Data Table

Activity Claim	Test Protocol	Contact Time	Test Organisms active against
Bactericidal Claim	EN13727 - dirty conditions	1 minute	<i>Ps. aeruginosa</i> , <i>E. coli</i> , <i>Staphylococcus aureus</i> , <i>Proteus vulgaris</i> , <i>Salmonella enterica</i> , <i>Enterococcus hirae</i> , <i>Acinetobacter baumannii</i> , <i>VRE faecium</i> , <i>Klebsiella pneumoniae</i>
Yeasticidal Claim	EN13624 - dirty conditions	1 minute	<i>Candida albicans</i>
Virucidal Claim	EN14476 - dirty conditions	30 seconds	Marine hepatitis virus (MHV)-1 for COVID-19



Manufactured by:
Pharmadesign Co., Ltd
199/3 Moo 21, Bangpleeyai
Bangplee Samutprakarn 10540
Made in Thailand

Developed in Australia by DentaLife Australia Pty Ltd
www.dentalife.com.au