



Clinicare

Hospital Grade Disinfectant

Pre-cleaning and Decontaminating Instructions

Antiviral

Anti-bacterial

Non-ionic

Non-streaking

Low Foaming



(Where appropriate, follow
Universal Precautions)

Pre-clean the surface with Clinicare Neutral Detergent or Hospital Grade Disinfectant Towelettes to remove all gross debris. Dispose of pre-cleaning wipe after decontamination. To disinfect the pre-cleaned instruments, use a fresh Clinicare Hospital Grade Disinfectant Towelette to thoroughly wet the surface and apply to surface and remain wet up to 3 minutes* at room (ambient) temperature. Clinicare Hospital Grade Disinfectant Towelettes effectively and rapidly kill germs on pre-cleaned instruments and medical devices previously soiled with blood or body fluids in healthcare or other critical care settings. Disinfects surfaces associated with the potential for transmission of infectious organisms.

(Where appropriate, follow
Universal Precautions)

Pre-clean the surface to be disinfected with Clinicare Disinfectant or Neutral Detergent to remove all gross debris. Dispose of pre-cleaning materials after decontamination. To disinfect pre-cleaned instruments or devices, use Clinicare Hospital Grade Disinfectant Solution to thoroughly wet the surface and apply to surface and remain wet up to 3 minutes* at room (ambient) temperature then wipe clean with a lint free cloth or towel. Clinicare Hospital Grade Disinfectant effectively and rapidly kill germs on pre-cleaned instruments and medical devices previously soiled with blood or body fluids in healthcare or other critical care settings. Disinfects surfaces associated with the potential for transmission of infectious organisms.

Do not mix with other chemicals or detergents. Use undiluted. Apply Clinicare Hospital Grade Disinfectant directly to pre-cleaned instruments and medical devices ensuring sufficient solution to wet the area to be disinfected. remain wet up to 3 minutes* before wiping off with a lint free cloth or towel. Allow surface to air dry before use. Discard any cloth or towel after use. Clinicare 5L solution can be used to refill Trigger Spray pack or other systems requiring Disinfectant Solution.

*Timing depends on organisms to be killed. See activity table.

Approved for use in all healthcare and critical environments.