



Clinicare

Hospital Grade IPA 70%

Pre-cleaning and Decontaminating Instructions

Broad spectrum

Anti-bacterial

Treats surfaces without leaving residues



(Where appropriate, follow
Universal Precautions)

Pre-clean the surface with Clinicare Neutral Detergent Towelettes to remove all gross debris. Dispose of any pre-cleaning wipes after decontamination. To disinfect the pre-cleaned instruments, use a fresh Clinicare IPA 70% Towelette to thoroughly wet the surface and allow to remain wet for 3 minutes at room (ambient) temperature. Clinicare IPA 70% Disinfectant Towelettes effectively and rapidly kill germs on pre-cleaned instruments or devices previously soiled with blood or body fluids in healthcare or other critical care settings. Disinfects surfaces associated with the potential for transmission of infectious organisms.

(Where appropriate, follow
Universal Precautions)

Pre-clean the surface to be disinfected with Clinicare Neutral Detergent to remove all gross debris. Dispose of pre-cleaning materials after decontamination. To disinfect pre-cleaned instruments or devices, use Clinicare IPA70% spray to thoroughly wet the surface. Allow to remain wet for 3 minutes at room (ambient) temperature then wipe clean with a lint free cloth or towel. Clinicare IPA70% effectively and rapidly kill germs on pre-cleaned instruments and medical devices previously soiled with blood or body fluids in healthcare or other critical care settings. Disinfects surfaces associated with the potential for transmission of infectious organisms.

Use undiluted. Do not mix with detergents or other chemicals. Discard solutions after use. Use gloves to avoid prolonged skin contact when handling solutions. Solutions may be sprayed or poured directly onto soiled surfaces. Clinicare 5L solution can be used to refill the Trigger Spray.

Approved for use in all healthcare and critical environments.